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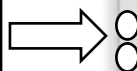
Stand: 02 | 2023

Assembly instructions HIIWI elevator

The assembly should be carried out by two people.

Required material:

- 4 wall plugs
- 4 screws, M8, min. 50 mm long
- Allen key

1

Unpack the lifting unit (lift). Place the lift in an upright position against the wall to which you wish to attach it. The second person should hold the lift in place to prevent it from tipping over.

2

Connect the mains plug to a wall socket. Using the remote control, move the lift all the way up so that the cover is fully visible.

3

Next remove the four outer screws and take off the cover. Now the wall bracket is exposed.

4

Hold the entire unit directly against the wall and mark the locations for the mounting holes in the wall through the four holes in the steel structure (diameter 8 mm). The lift can be precisely aligned using the height-adjustable feet. This requires an Allen key.

5

Put the lift aside and at the four marked locations drill the holes in the wall. For wall mounting, four screws with a length of at least 50 mm and sufficient load-bearing capacity are required as well as corresponding wall plugs depending on the substrate (wall construction).

6

Place the lift back into position and screw it to the wall.

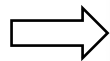
7

Replace the cover and tighten it with the four screws previously removed. Now you can move the support downwards.

8

In the next step attach the foldable changing table to the wall hanging plate. Unfold the changing table and screw in the safety screws. Please make sure that the changing table is firmly and securely attached. This completes the assembly.

Attachment of the remote control for people using wheelchairs
Please attach the 2 Velcro points at a handle height of 80 -85 cm

version: 02 | 2023